



Approved products and methods for cleaning and disinfecting of OR Table pads CAT# **398-LG** OR Table Pads and **398-ABG** Arm Board Pads sold by Innovative Medical Products.

DO NOT APPLY TAPE TO ANY PAD

CLEANING OF SOILS & STAINS

Wash with a generous application of neutral soap suds and lukewarm water up to 103 Degrees F. Rinse with water and dry. Do not immerse. or

Use a diluted bleach solution of 10-parts water to 1-part bleach. Wipe dry with a clean cloth. Do not immerse. Bleach **MUST** be diluted to the 10:1 ratio.

All chemicals/solutions **MUST** be diluted to the manufacturers recommendations (look closely, they often have the proper dilutions in “fine print” on the container – they **WANT** you to use more than you need to). No chemical or solution should be left to dry on the pad surface. After the allotted time has passed to insure disinfection, the pad surface should be wiped again with a wet cloth to remove the remaining chemical or solution.

DISINFECTING

Use Lysol Brand III I.C. Disinfectant Aerosol Spray by Reckitt Benckiser Inc., Montvale, NJ -- per manufacturer’s instructions.

Reference: Reckitt Benckiser UPC # 36241-95029-3

or

Use a diluted bleach solution of 10-parts water to 1-part bleach. Wipe dry with a clean cloth. Do not immerse. Bleach **MUST** be diluted to the 10:1 ratio.

All chemicals/solutions **MUST** be diluted to the manufacturers recommendations (look closely, they often have the proper dilutions in “fine print” on the container – they **WANT** you to use more than you need to). No chemical or solution should be left to dry on the pad surface. After the allotted time has passed to insure disinfection, the pad surface should be wiped again with a wet cloth to remove the remaining chemical or solution.

The following Steris products are also approved for use in cleaning and disinfecting our pads:

- Steris Coverage Plus wipes 1608-WC, -GS, -G4
- Steris Coverage Spray HB Plus 1624-77
- Steris Coverage Spray TB Plus 1629-B4, -08

The use of ANY cleaning/disinfecting chemical other than those on this list will void the warranty on any Table Pads sold by IMP.