



imp De Mayo Knee & De Mayo D² Knee Positioner® “Spring Loaded” Stainless Steel Handle Instructions For Use

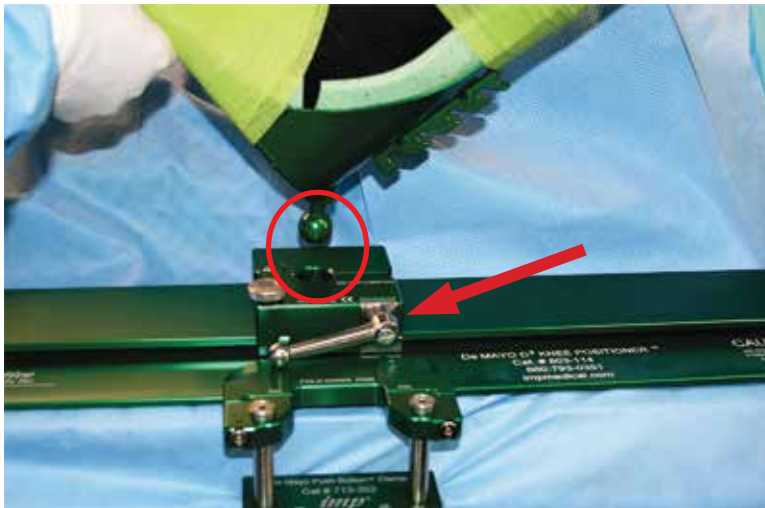

1. With the Handle all the way to the Left (counterclockwise), place the patient's foot and boot into the carriage.

Turning the E-Brake clockwise until tight will prevent the carriage from moving.

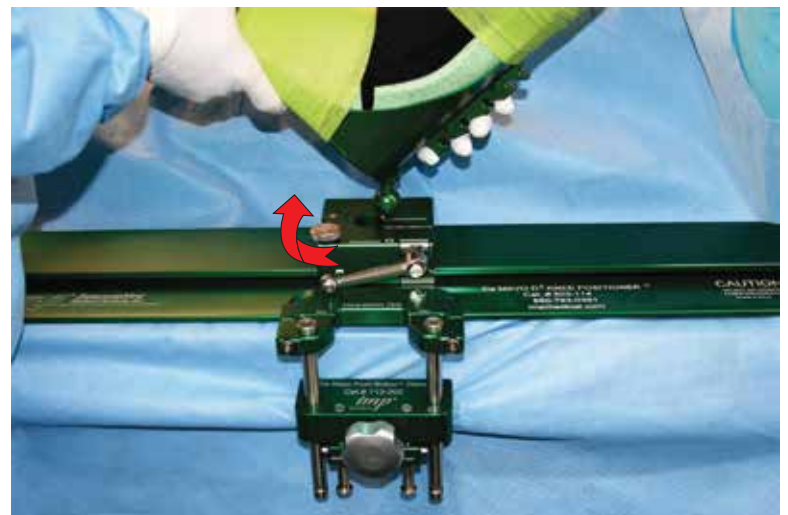




2. *Rotate* the Handle to the Right (clockwise),


until it is finger tight.




3. To reposition the Handle while in the locked position, pull the Handle towards you creating a "gap" and while pulling on the Handle,

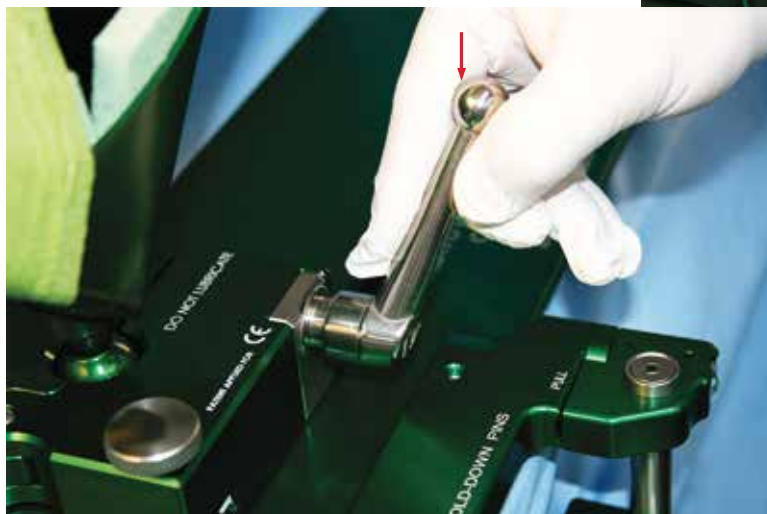

rotate the Handle counterclockwise to the 2:00 o'clock position and release.

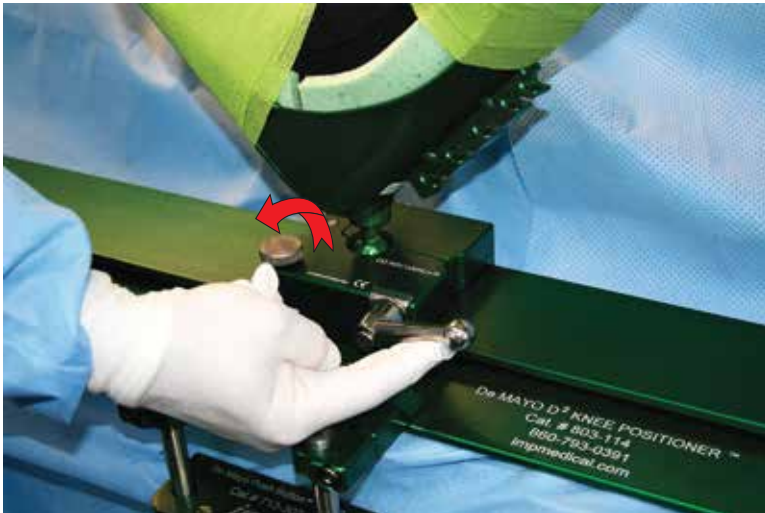

4. To unlock the carriage, turn the E-Brake counterclockwise, then *rotate* the stainless steel Handle all the way to the Left.

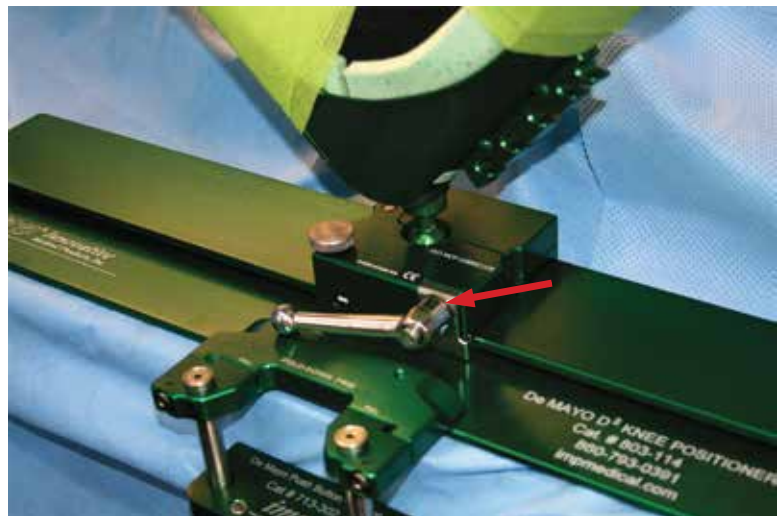

5. If the carriage has been over tightened, pull the stainless steel Handle towards you,


and while pulling, rotate the Handle clockwise half a turn to the Right



and release the Handle.



NOTE: Follow Step 4 to unlock the Handle and Carriage.

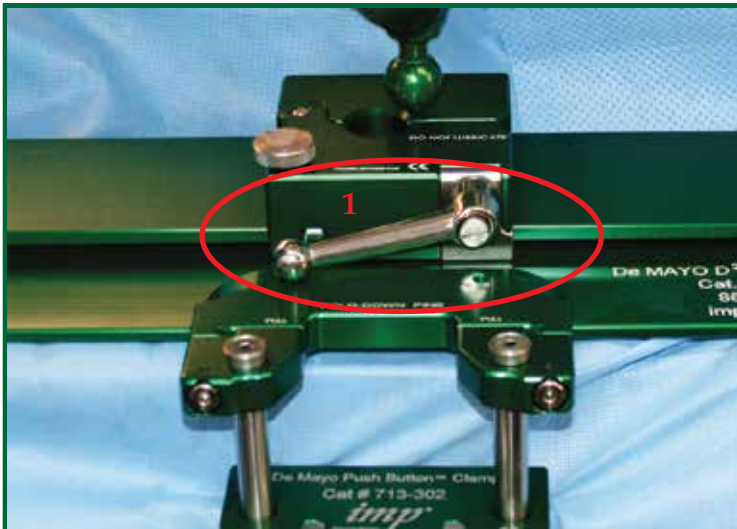

Set Up Test Procedure

1. Check to ensure Handle will lock the carriage once the boot is inserted by rotating it clockwise. Check to ensure the Handle will release the boot by rotating it counterclockwise.

