


# *imp* De Mayo V<sup>2</sup> Knee Positioner Carriage Instructions For Use

## Removal and Disassembly of the V<sup>2</sup> Carriage

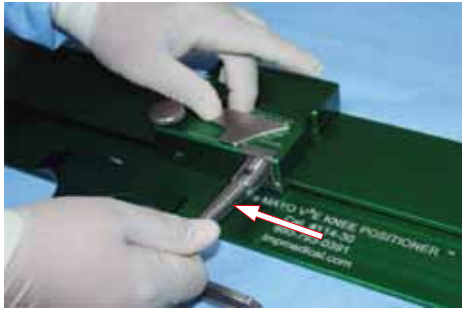

1. Straighten and turn the handle of the carriage counter clockwise to loosen enough to create a "gap" between the handle plate and carriage.





2. Slide the carriage toward the end of the base plate with the CE mark. To remove the carriage, pivot it around the curve of the base plate.



3. Completely remove the handle. Open the carriage and pivot the teflon pad open.




4. Remove the E-Stop by turning it counter clockwise and slide the varus tilt bar off. Proceed with cleaning.

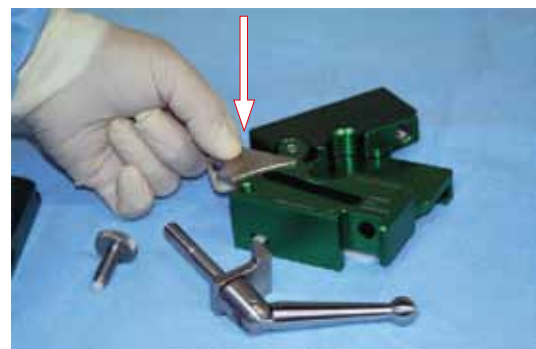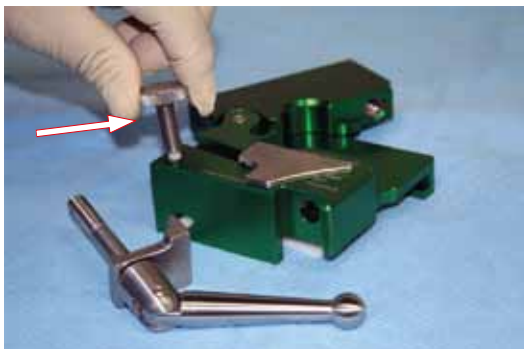

## Assemble and Attach V<sup>2</sup> Carriage




1. Insert the varus tilt bar and slide it all the way to the Open position. Insert the E-Stop and screw it in ½ of the way.

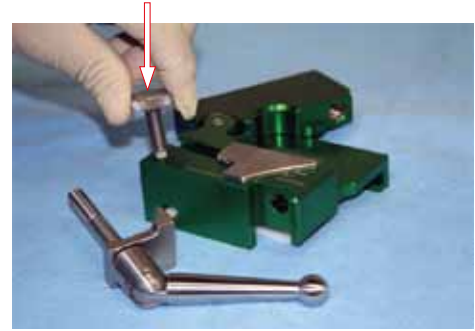

2. Pivot the teflon pad up and while holding the pad, insert the handle, orienting the "L" foot of the handle plate toward the pad. Leave a gap between the handle plate and carriage.

3. Slide the carriage onto the CE marked side of the base plate with the handle on the CE side. Pivot the carriage around the curve of the base plate and slide it onto the base plate.

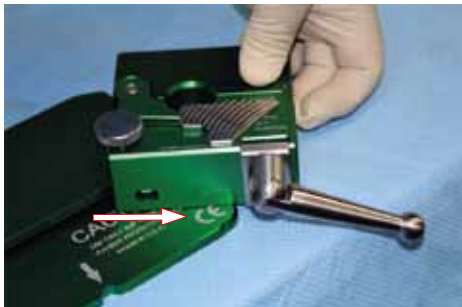

## Correct Handle Position

Insert the boot into the carriage and "turn" the handle clockwise (straight as a screwdriver) until it is finger tight. Pivot the handle and rotate with one finger to the final position (as illustrated).

