

imp De Mayo Universal Distractor® with the De Mayo Knee Positioner® System Set Up Instructions

1. Replace OR Table Pads with green *imp* pads.



If using X-Ray cassette,
it is not necessary to remove
the X-ray cassette plates.

2. Position patient with gluteal fold at separation of pads.



3. During prep of surgical leg, remove single OR Table Pad.



4. Create a 3"- 5" fold in final drape under the buttocks.



5. Place De Mayo Knee Positioner® in the well
to achieve maximum flexion.



imp De Mayo Universal Distractor® with the
De Mayo Knee Positioner® System Set Up Instructions Continued- page 2

6. Check flexion for final approval of freedom of the drape to allow full range of motion when the Positioner is locked on drapes.



7. Position clamp directly under gliding pins and lock to the OR Table Side Rail with the Single Lever Clamp.




ATTACHING THE SINGLE LEVER CLAMP



8. Position the clamp under the pins. With Left hand, squeeze the jaws tight and rotate the lever to lock.



9. Holding the jaws, tighten the knurled knob all the way.

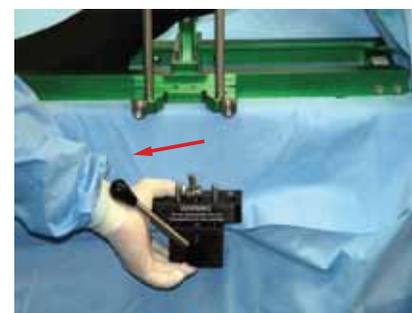


10. While holding the clamp, rotate the lever to unlock.



11. Turn the knurled knob 1/4 to 1/2 a rotation.




12. Rotate the lever to lock position requiring only 2 finger pressure. Works like a vice grip and “snaps” into place.

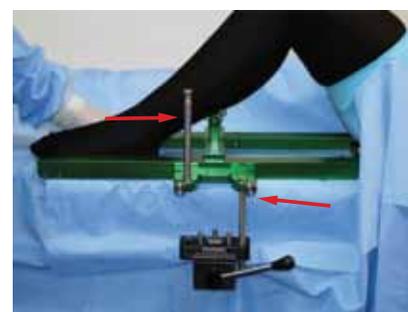

13. Slide pins into clamp.

14. Tighten knurled knobs.

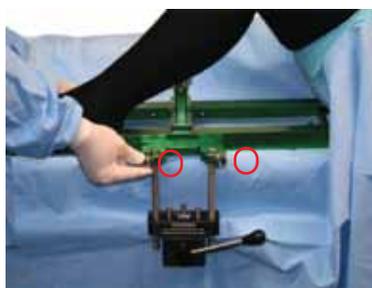

imp De Mayo Universal Distractor[®] with the
De Mayo Knee Positioner[®] System Set Up Instructions Continued- page 3

15. Protect your patient from potential Pressure Ulcers. Insert the *imp* Patient Protector Pad into the sterile boot.



16. Wrap cohesive bandage around the foot starting with a minimum of six (6) foot wraps, tear, and finish the wrap above the bracket around the calf.

Avoid wrapping cohesive bandage over the bracket on the back of the boot




17. Place the Pressure Protector Pad on the square post.



18. With the leg at 90° of flexion, position the fully collapsed Distractor placing the pressure Protector Pad at the distal femur.



19. Insert the small pin at the base of the Distractor in the most Proximal slot on the back of the boot.

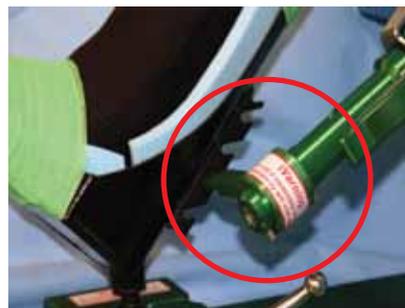

imp De Mayo Universal Distractor® with the
De Mayo Knee Positioner® System Set Up Instructions Continued- page 4

20. In addition to the Single Locking Lever, as greater forces are applied, the thumb screw “Emergency Brake” provides extra locking pressure when using the Distractor.



21. Squeeze the large green handle to *distract*.



22. Press the small silver lever to *release* distraction.



*Set Up for the
De Mayo Distractor®
with the
De Mayo Knee Positioner®
is now complete*



De Mayo Universal Distractor® is a registered trademark of Innovative Medical Products, Inc.
De Mayo Universal Distractor® U.S. Patent No. 8,048,082
De Mayo Knee Positioner® is a registered trademark of Innovative Medical Products, Inc.
De Mayo Knee Positioner® U.S. Patent No. 7,380,299
©2013 Innovative Medical Products, Inc.
ALL RIGHTS RESERVED 01/2013

imp® **Innovative**
Medical Products, Inc.
87 Spring Lane, PO Box 8028
Plainville, CT 06062
PH: (860) 793-0391 FAX: (866) 459-1805
Toll Free PH: 1-800-467-4944
www.impmedical.com