



HAND CLEANING INSTRUCTIONS



TURNSTILE CAST STAND CAT # 1091-B

THE TURNSTILE CAST STAND MAY BE DISINFECTED, AS A RULE WITH ANY HOSPITAL GRADE DISINFECTANT.

THESE INCLUDE BUT ARE NOT LIMITED TO:

HI-TOR PLUS

BEAUCOUP GERMICIDAL DETERGENT

SANI MASTER II OR III

STARLINE INSTRUMENT DETERGENT.

A SOLUTION OF WATER AND COMMON DETERGENT MAY BE USED TO CLEAN AND DISINFECT.

COMPLETELY DRY THE TURNSTILE CAST STAND PRIOR TO NEXT USE.