

imp **Humbles LapWrap® Positioning Pad** **Instructions For Use**



1. Place LapWrap® on OR table pads, VELCRO® brand black hook fastener side UP.


2. Place patient so the upper edge of the LapWrap® is located in the axillary region.



3. Wrap entire 3 piece strap around the patient's arm and attach VELCRO® brand hook-and-loop fastener near or under patient.



4. For unusual table positions, patient can be secured to the table by placing center tongue between OR table and side rail. Wrap tongue around side rail and attach VELCRO® brand black hook strap to VELCRO® brand green loop.

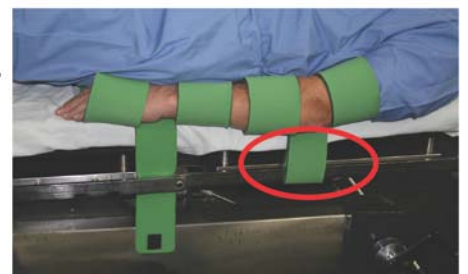

imp **Humbles LapWrap® Positioning Pad** **Instructions For Use**

FOR BARIATRIC PATIENTS:



1. Lift LapWrap® center tongue of the upper arm and pull between the torso and the biceps.



2. Insert the tongue out through the cutout, while pulling the VELCRO® brand hook-and-loop fastener around the outer arm and towards the body.

3. Fasten to the center tongue.



Optional Technique:



Place TUFFease® pressure redistribution pad on OR table for extra patient protection.

UNIVERSAL DESIGN
FITS ALL SIZE PATIENTS



Cat.#: 904-G
10/Case

Set Up Test Procedure



1. Check that Black Hook is securely attached to Green Loop Material at all locations.



LapWrap® is a registered trademark of Innovative Medical Products, Inc.
LapWrap® - U.S. PATENT No. 8,001,635
TUFFease® is a registered trademark of Innovative Medical Products, Inc.
VELCRO® is a registered trademark of Velcro Industries B. V.
© August 2014 Innovative Medical Products, Inc.
ALL RIGHTS RESERVED

imp **Innovative**

Medical Products, Inc.
87 Spring Lane, PO Box 8028
Plainville, CT 06062
PH: (860) 793-0391 FAX: (866) 459-1805
Toll Free PH: 1-800-467-4944
www.impmedical.com