

# De Mayo RoTractor®


## Instructions For Use ~ *Non-Sterile* Application




1. Attach the RoTractor® end to the adaptor with the receiver hole facing the locking knob.




2. Tighten the locking knob of the adaptor.

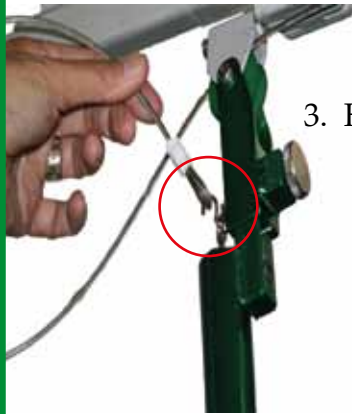

3. Hook traction wire to eyelet in RoTractor® extension.

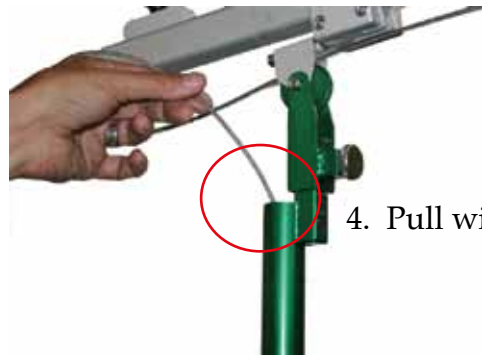

4. Pull wire into extension.

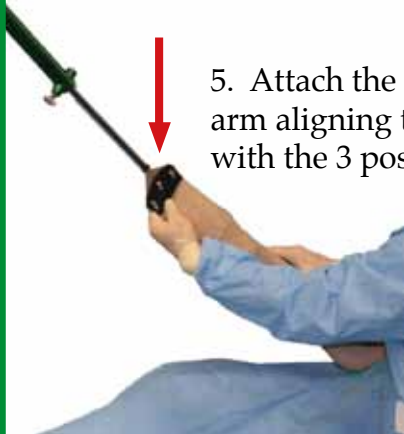

5. Attach the Phase 4 Gel™ Splint with the patient's arm aligning the 3 holes at the end of the splint with the 3 posts of the RoTractor® plate.

### Set Up Test Procedure



Tighten to Lock Knobs securely:

- 1 - RoTractor® Knob & Connection
- 2 - Check the slide of the RoTractor® and Rotation Locking Knob
- 3 - Check the Connection with the Gel Splint



A DIVISION OF INNOVATIVE MEDICAL PRODUCTS, INC.

87 Spring Lane, PO Box 8028

Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944

www.impmedical.com

De Mayo RoTractor® is a registered trademark of Innovative Medical Products, Inc.  
De Mayo RoTractor® - PATENT APPLIED FOR  
Phase 4 Gel™ is a trademark of Innovative Medical Product, Inc.  
Phase 4 Gel™ Splint - PATENT APPLIED FOR  
ALL RIGHTS RESERVED  
© February 2014 ism® / Innovative Medical Products, Inc.