



STERILIZATION CASE:

1109-XS, DeMayo Ankle Distractor Accessories

1109-S, DeMayo Ankle Distractor Base & Bridge

Cleaning & Sterilization Procedure

NOTE: ALL SOLUTIONS MUST BE COMPATIBLE WITH ALUMINUM & STAINLESS STEEL.

Recommended Hand Cleaning Instructions.

- Soak the product in an enzyme solution. Follow the manufacturer's direction for dilution and soaking time.
- Put thru washer / sterilizer cycle, continue as normal procedure.
- Rinse the product in warm tap water.
- Soak or wipe the product down with a hospital approved and / or EPA approved germicide.
- Rinse the product in warm tap water.
- Wash the product with an instrument detergent up to a pH of 9.0
- RINSE THE PRODUCT IN WARM TAP WATER.
- DRY THOROUGHLY AND WRAP ACCORDINGLY:
48" x 48" WRAP. (1109-XS) 54" x 54" WRAP (1109-S)

Recommended Sterilization Instructions.

- Insure that all parts are thoroughly cleaned.
Make sure that all movable parts are loose and can move freely.
- DO NOT PLACE ANKLE DISTRACTOR IN A MILK BATH OR LUBRICATE.
- Place Ankle Distractor and/or accessories into Sterilization Case.

Double wrap in two disposable or reusable wraps.

Run normal vacuum cycle for your institution.

Minimum recommended parameters are as follows:

Pre-Vac Sterilization

EXPOSURE TIME:	4 MINS
TEMPERATURE:	270* - 272* F 133* - 134* C
DRYING TIME:	20 MINUTES
COOLING:	@ ROOM TEMPERATURE