



## HAND CLEANING INSTRUCTIONS

- **UNIVERSAL LATERAL HIP POSITIONER.**
- **De MAYO HIP POSITIONER.**
- **MORPHBOARD POSITIONING SYSTEM.**

THE FAMILY OF IMP HIP POSITIONERS AND ALL OF THE POSITIONER'S COMPONENTS CAN BE CLEANED IN THE FOLLOWING MANNER:

1. COMPONENTS MAY BE DISINFECTED, AS A RULE WITH ANY HOSPITAL GRADE DISINFECTANT TO REPROCESS THE COMPONENTS BETWEEN CASES.

THESE INCLUDE BUT ARE NOT LIMITED TO:

HI-TOR PLUS

BEAUCOUP GERMICIDAL DETERGENT

SANI MASTER II OR III

STARLINE INSTRUMENT DETERGENT.

2. A SOLUTION OF WATER AND LIQUID BLEACH (10 PARTS WATER TO 1 PART BLEACH ) MAY ALSO BE USED TO CLEAN AND DISINFECT.

THOROUGHLY CLEAN THE COMPONENTS WITH YOUR DISINFECTANT. A FINAL WIPE DOWN WITH CLEAN WATER IS RECOMMENDED TO BE SURE THAT NO RESIDUE FROM THE DISINFECTANT IS LEFT ON THE COMPONENTS.

COMPLETELY DRY THE COMPONENTS AND RETURN THEM TO THE STORAGE CASE TO PROTECT THEM FROM DAMAGE.

3. IMP RECOMMENDS THAT ALL COMPONENTS FOR EACH SYSTEM BE KEPT TOGETHER AS A KIT AND A COMPONENT NOT BE USED WITH ANOTHER KIT. ALL COMPONENTS ARE TESTED AS A KIT BEFORE SHIPPING.