

Estate

TrenMAX®

Trendelenburg Positioner
The Sticky Pad™

Armboard Extension Instructions For Use



Ensure that the board extensions are inserted between the white straps.

1. Insert the TrenMAX® Armboard at an angle which will clear any possible underside stitching of the table pad.




2. While inserting the Armboard, an assistant may be required to hold the patient's arms to slightly roll the patient away from the edge.



3. Seat the Armboard until it makes contact with the patient's arm or the white straps.



4. The distal end should cover the wrist and hand.

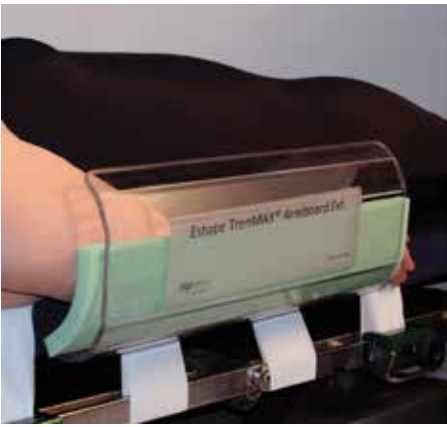

5. Place some padding to protect the patient. The type of padding is at the discretion of the facility.
[Padding not available from IMP]



Safety Hook & Loop Strap with Buckle provided with Armboard Extension.

6. Attach Safety Hook & Loop Strap.



Slide safety strap through the 2 slots on Armboard. Wrap buckle end around table side rail. Feed rounded end through buckle, and attach hook end to loop.



Set Up Test Procedure



1 ~ Check that Armboard Extensions are between straps.

2 ~ As applicable, confirm padding is protecting patient's arm.

3 ~ Check Armboard Safety Strap is attached & secured to table side rail.



TrenMAX® is a registered trademark of Innovative Medical Products, Inc.

PATENT APPLIED FOR

TrenMAX® Clamp ~ PATENT APPLIED FOR

The Sticky Pad™ is a trademark of Innovative Medical Products, Inc.

© June 2017 ALL RIGHTS RESERVED

imp Innovative

Medical Products, Inc.

87 Spring Lane, PO Box 8028

Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944

www.impmedical.com