


Estape


TrenMAX™

Trendelenburg Pad

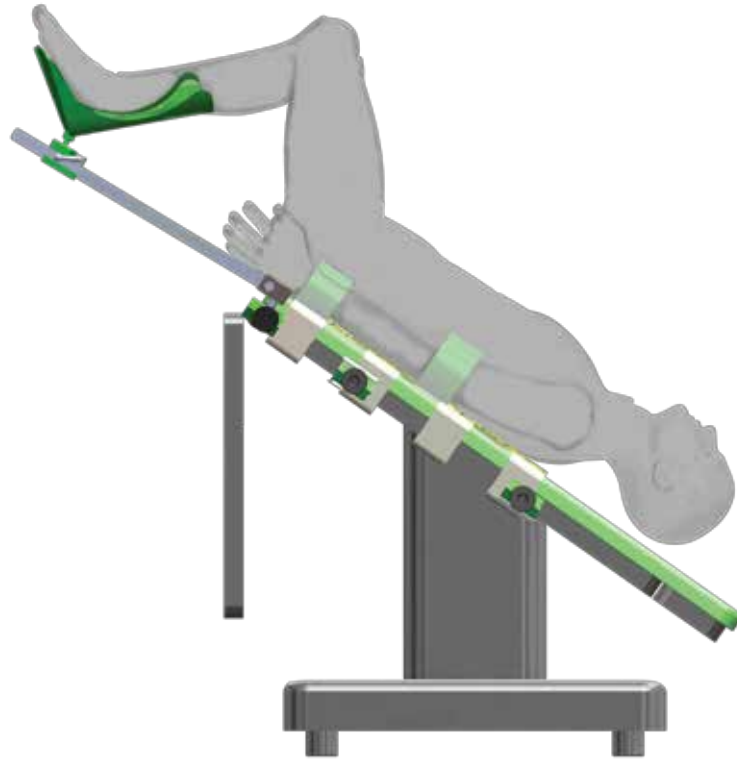

Instructions For Use

Trendelenburg Technique

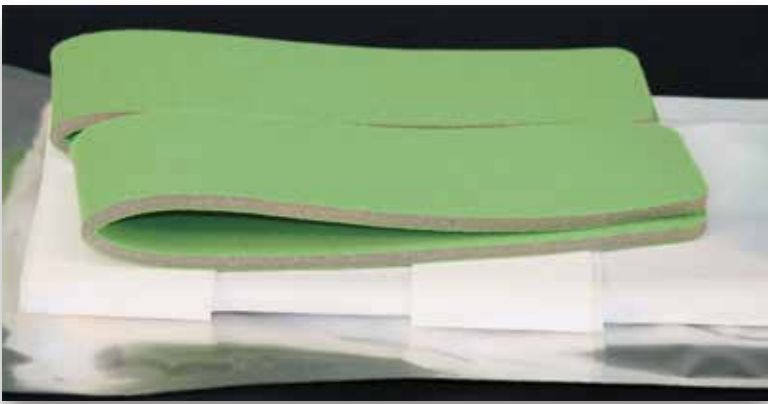

1. Remove the pads from the package and place the Green Pads on a back table.

2. Open the White Base Pad and position the distal edge one (1) handbreadth (or approximately six (6) inches) from the peroneal cutout leaving space on the side rail to attach the stirrups.




3. Position the White Straps between the OR table and the side rail.




4. Lift the "Gel" Pad exposing the Black Hook Material and attach the White Strap to the lower Black Hook.




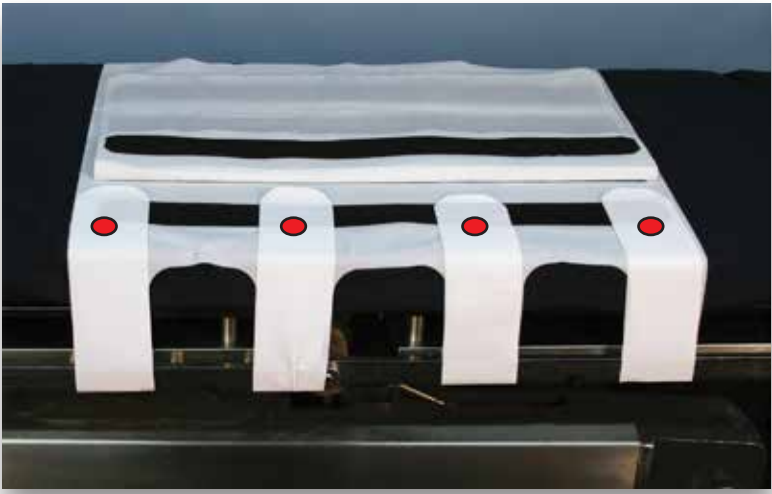


5. Attach all four (4) Straps. ●

6. Attach all four (4) Straps on the other side by pulling and tensioning the Strap indenting the table pad, then pull the White Strap up and attach to the lower Black Hook, proceed to the other side.

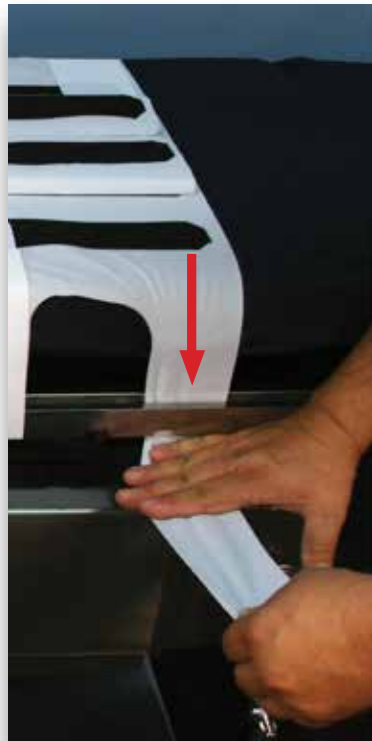

Lower the top flap on both sides

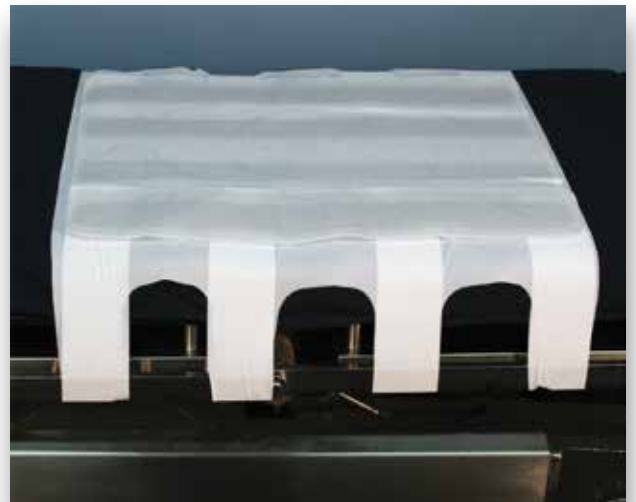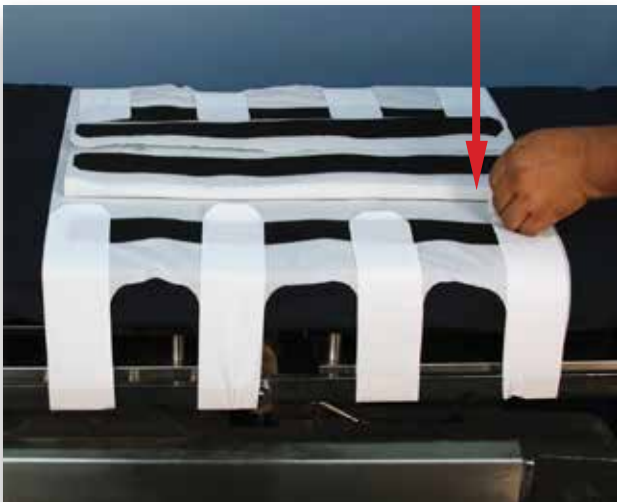




7. On the opposite side from the patient transfer, attach the TrenMAX™ Clamp over the proximal Strap, by attaching Clamp over one side of the strap and then the other.



8. Seat the clamp jaws over the Strap and side rail while holding the Pivot Bar, then seat the Pivot Bar over the side rail and tighten Knob.


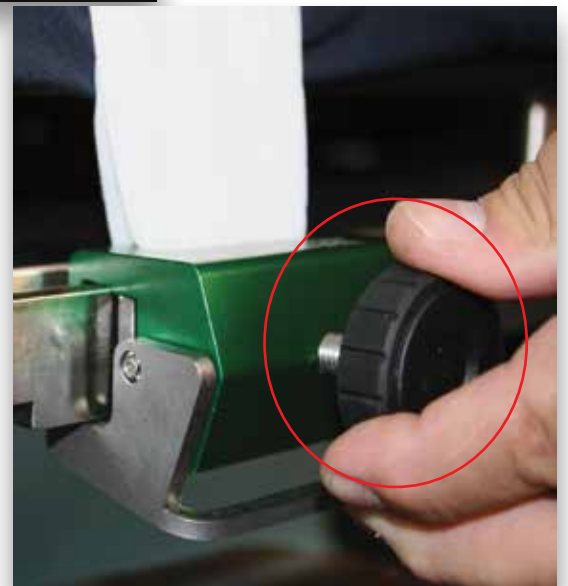


9. Attach another TrenMAX™ Clamp to the distal Strap on the same side.




10. Transfer the patient on the side without the Clamps using a draw sheet.

Move the patient towards the end of the OR table with the draw sheet for Robotic procedures.

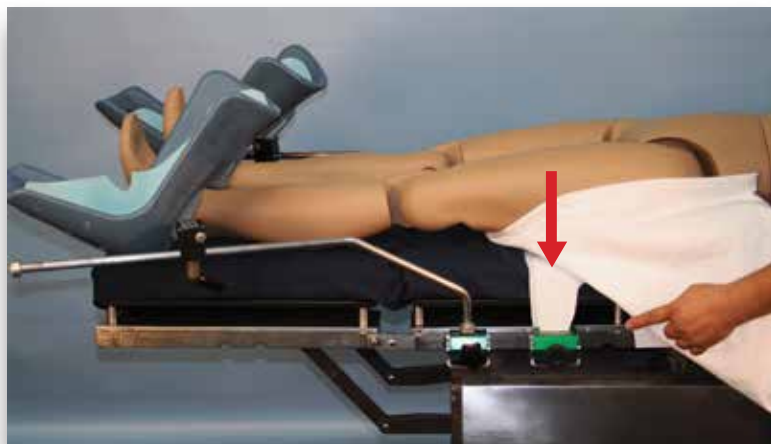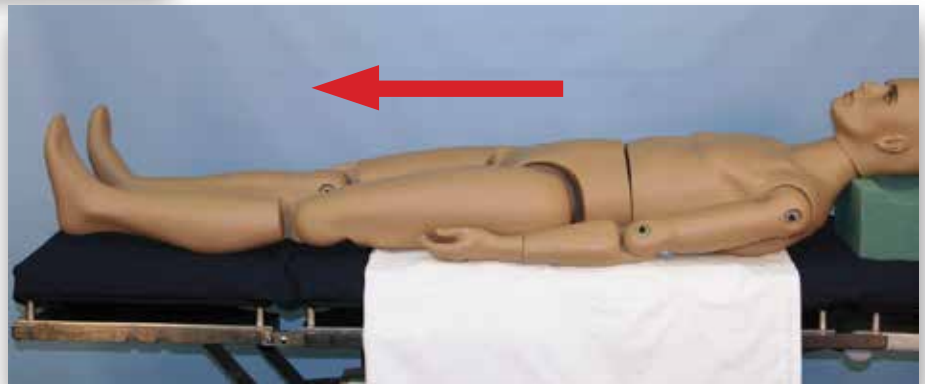

11. Attach the stirrups then the TrenMAX™ Clamps to the distal and proximal White Straps, and remove draw sheet.

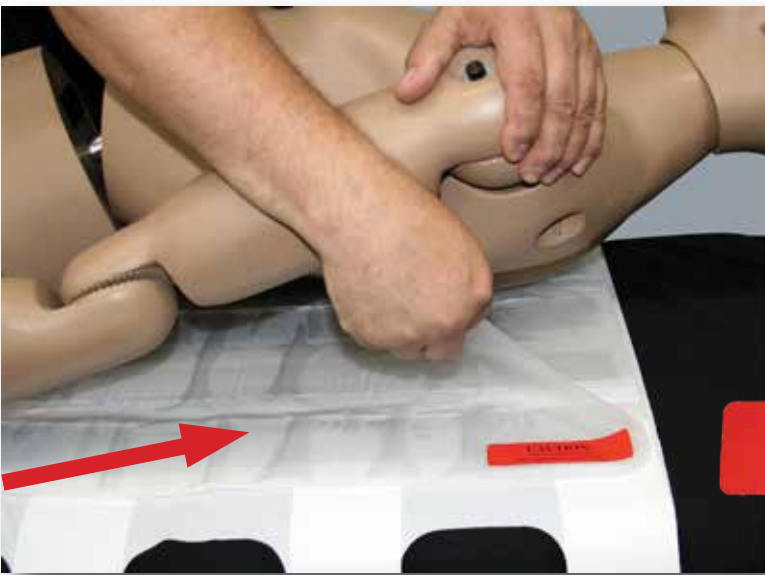

12. Lift the patient's shoulder and peel the Protective Plastic Sheet off the Gel. Repeat on the other shoulder.

CAUTION:
Peel the plastic sheet off
after removing the transfer sheet.



13. Lift the Gel Pad and Hook and attach the Green Foam Arm Strap to the lower Black Hook, positioning the two (2) Straps proximal and distal to the elbow.




14. Loop the Green Foam Arm Strap around the arm and attach it to the Black Hook Material.



15. Attach all four (4) Green Foam Arm Straps in the same fashion.

16. Proceed to positioning the patient in Trendelenburg.

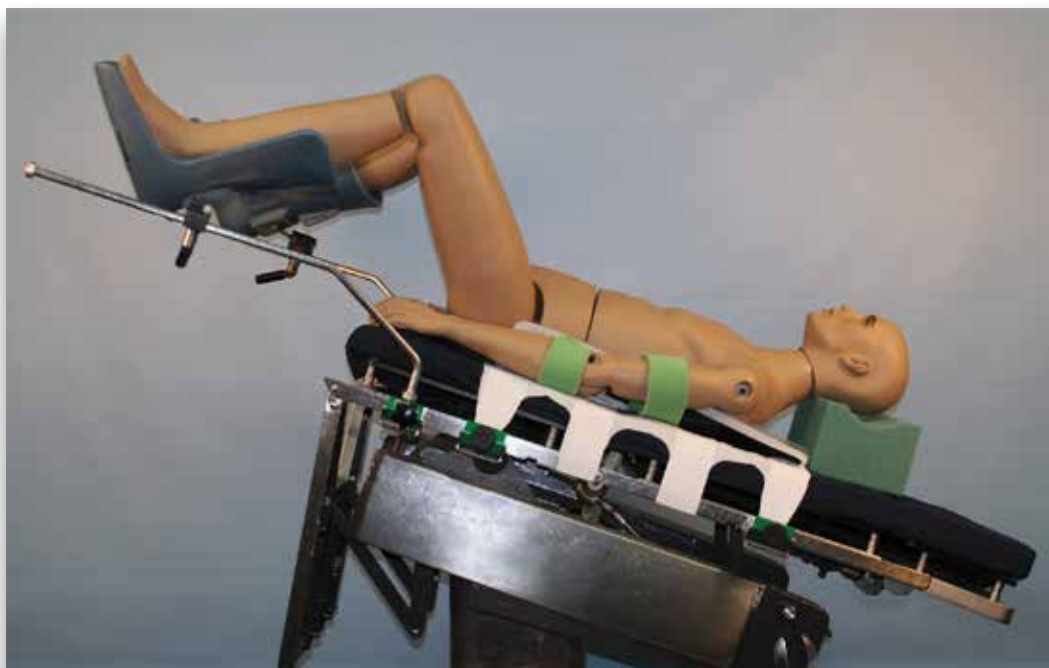


Estape
TrenMAX™
Trendelenburg Pad




At conclusion of the procedure:

To transfer the patient from the OR table, simply cut the four (4) Green Foam Arm Straps.

To remove the Pad, simply cut the four (4) White Straps.

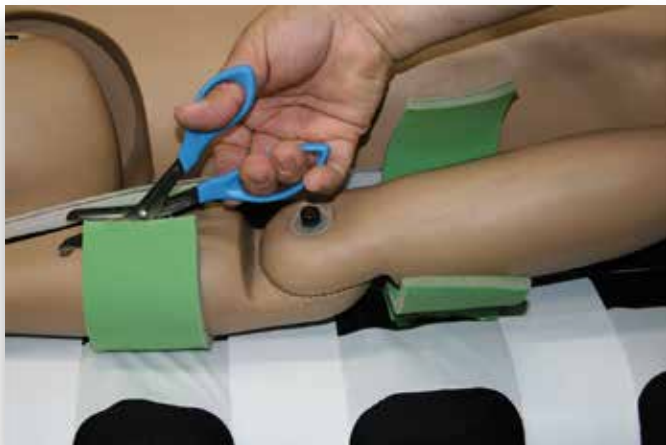

Set Up Test Procedure

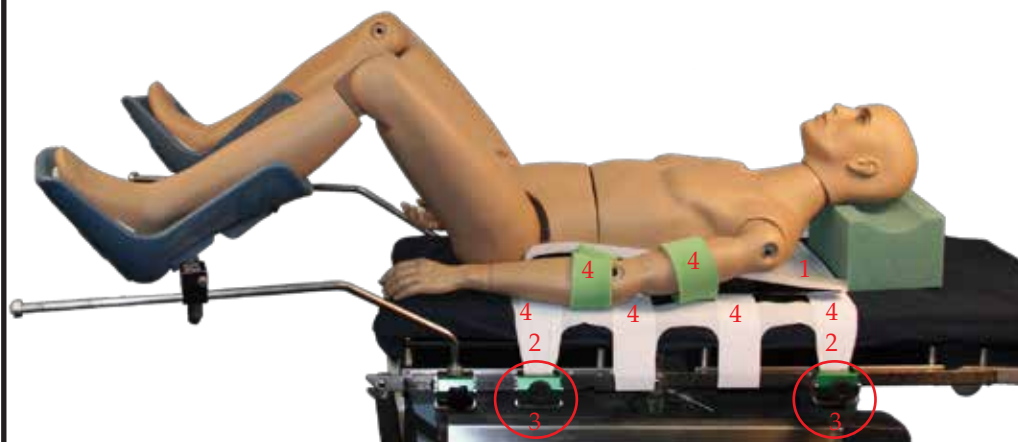

1 ~ Ensure that the Protective Plastic Sheet has been removed exposing sticky gel.

2 ~ Check that white Safety Straps are taut and there are a minimum of 2 TrenMAX™ Clamps on each side of the table.

3 ~ TrenMAX™ Clamps are tightened with Locking Knobs.

4 ~ Check to ensure all hook and loop straps are secure.



Designed in conjunction with Ricardo Estape, M.D. Miami FL
 TrenMAX™ is a trademark of Innovative Medical Products, Inc.
 PATENT APPLIED FOR
 TrenMAX™ Clamp ~ PATENT APPLIED FOR
 Phase 4™ Gel is a trademark of Innovative Medical Products, Inc.
 © May 2016 Innovative Medical Products, Inc.
 ALL RIGHTS RESERVED

imp® Innovative
 Medical Products, Inc.
 87 Spring Lane, PO Box 8028
 Plainville, CT 06062
 PH: (860) 793-0391 FAX: (866) 459-1805
 Toll Free PH: 1-800-467-4944
www.impmedical.com