

imp **Universal SteriBump®**

Instructions For Use



1. Hold package at the edge and lift the base of the white sealed paper.







2. In one motion, peel the white seal off the opening of the sterile package.



3. Present the SteriBump® to the sterile field with appropriate sterile technique protocol.



4. Unwrap the SteriBump®,

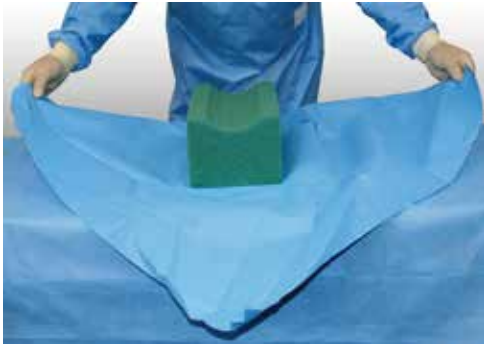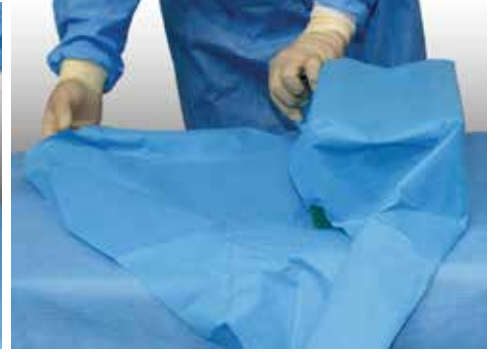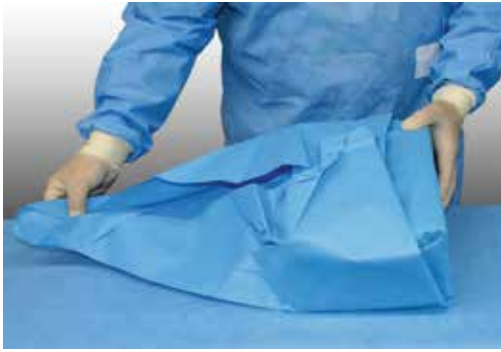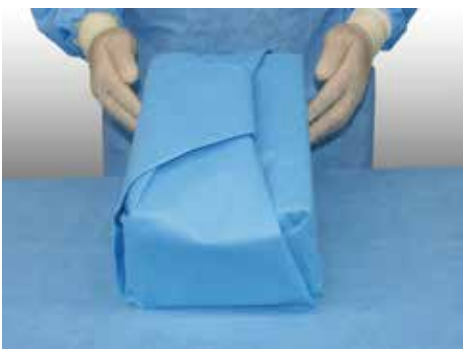

and place the SteriBump® under the desired limb per procedure. Curved side of the SteriBump® towards extremity.



Set Up Test Procedure



1. Place *flat* side of the SteriBump® on a flat surface.

2. *Curved* side of the SteriBump® toward patient's extremity.



STERILE EO

imp Innovative

Medical Products, Inc.

87 Spring Lane, PO Box 8028

Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944

www.impmedical.com